

Alta2 | Nera2 | Ria2

INSTRUCTIONS FOR USE

BTE13 PP



oticon
PEOPLE FIRST

Introduction to this booklet

This booklet guides you in how to use and maintain your new hearing instrument. Please read the booklet carefully including the **Warning section**. This will help you to achieve the full benefit of your new hearing instrument.

Your Hearing Care Professional has adjusted the hearing instrument to meet your needs. If you have additional questions, please contact your Hearing Care Professional.

| **About** | Start up | Handling | Options | Warnings | More info |

For your convenience, this booklet contains a navigation bar to help you navigate easily through the different sections.

Intended use

The hearing instrument is intended to amplify and transmit sound to the ear and thereby compensate for impaired hearing. This power hearing instrument is intended to be used by adults and children (>36 months) with moderate-to-severe up to severe-to-profound hearing loss.

IMPORTANT NOTICE

The hearing instrument amplification is uniquely adjusted and optimized to your personal hearing capabilities during the instrument fitting performed by your Hearing Care Professional.

Table of content

About	Identify your hearing instrument style and ear piece	6
	Sound hook and mold	8
	Thin tube (Corda miniFit)	10
	Identify left and right hearing instrument	12
	Battery (size 13)	13
Start up	Turn the hearing instrument on and off	14
	When to replace the battery	15
	How to replace the battery	16
Handling	Caring for your hearing instrument	18
	Sound hook with mold	20
	Put on the hearing instrument with mold	21
	Clean the mold	22
	Wash the mold	23
	Thin tube (Corda miniFit)	24
	Put on the hearing instrument with thin tube	25
	Clean the thin tube	26
	Thin tube with dome	28


	Replace the dome	29
	Thin tube with Corda mold	30
	Clean the Corda mold	31
Options	Optional features and accessories	32
	Mute the hearing instrument (optional)	33
	Change volume (optional)	34
	Change programs (optional)	36
	Other options (optional)	38
	Wireless accessories (optional)	40
	Tamper-resistant battery drawer (optional)	42
Warnings	Warnings	44
More info	Troubleshooting guide	54
	Water resistant (IP58)	56
	Conditions	57
	Warranty certificate	58
	International warranty	59
	Mobile phone	60
	Technical information	62
	Technical Data	67

Identify your hearing instrument style and ear piece

There are two different attachments and several different ear pieces which classify your hearing instrument style.


Please identify your attachment and ear piece. This will make it easier for you to navigate through this booklet.

- Sound hook and mold**
An instrument with a sound hook will have a mold as an ear piece.

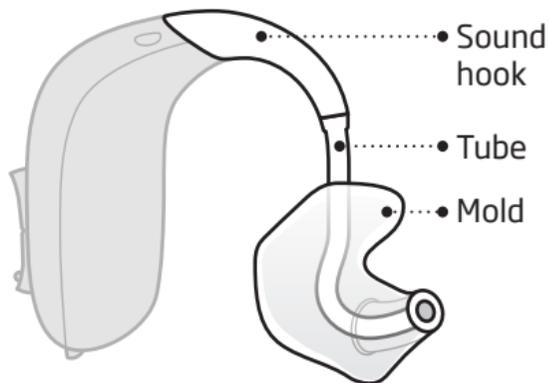

□ **Thin tube (Corda miniFit)**

An instrument with a thin tube will have one of the following ear pieces: dome or Corda mold.

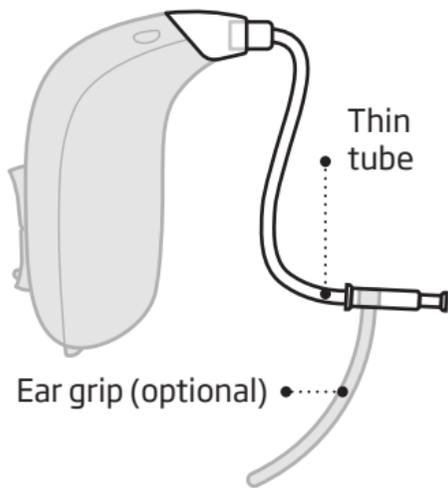


Dome




Corda mold

Sound hook and mold

What it is

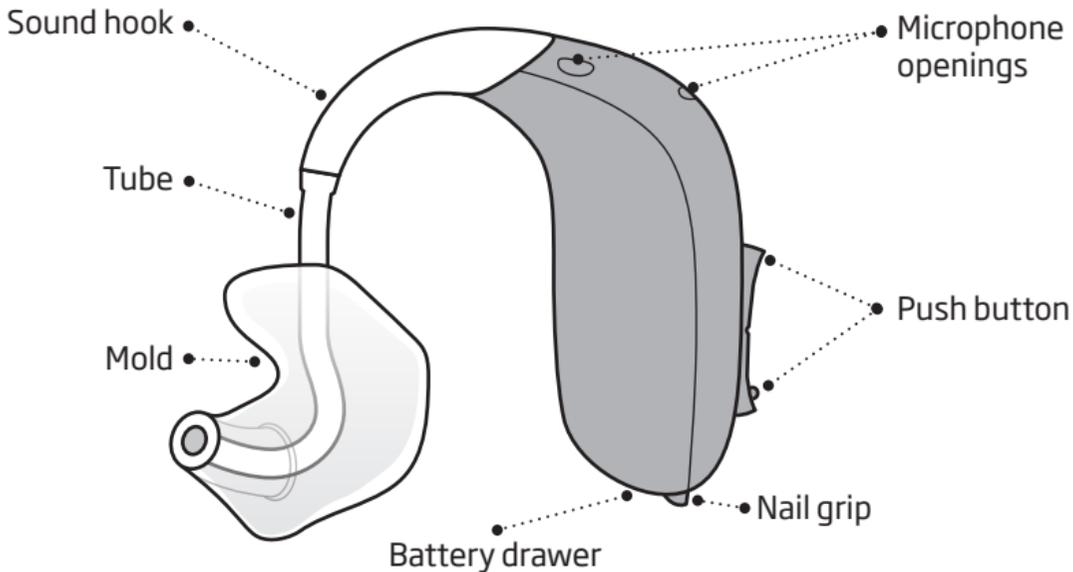


What it does

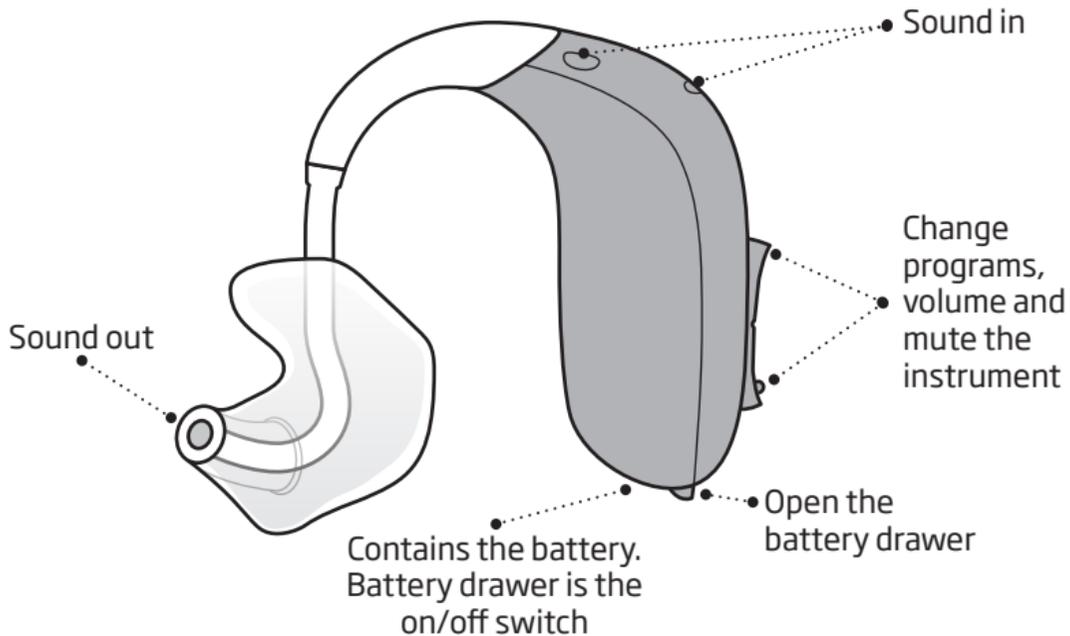


Thin tube (Corda miniFit)

What it is

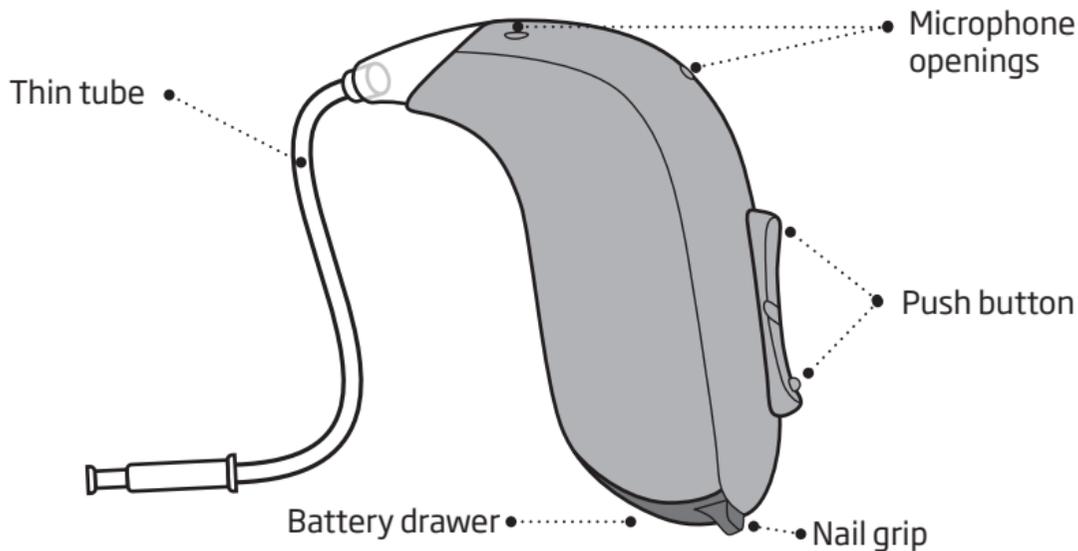


What it does

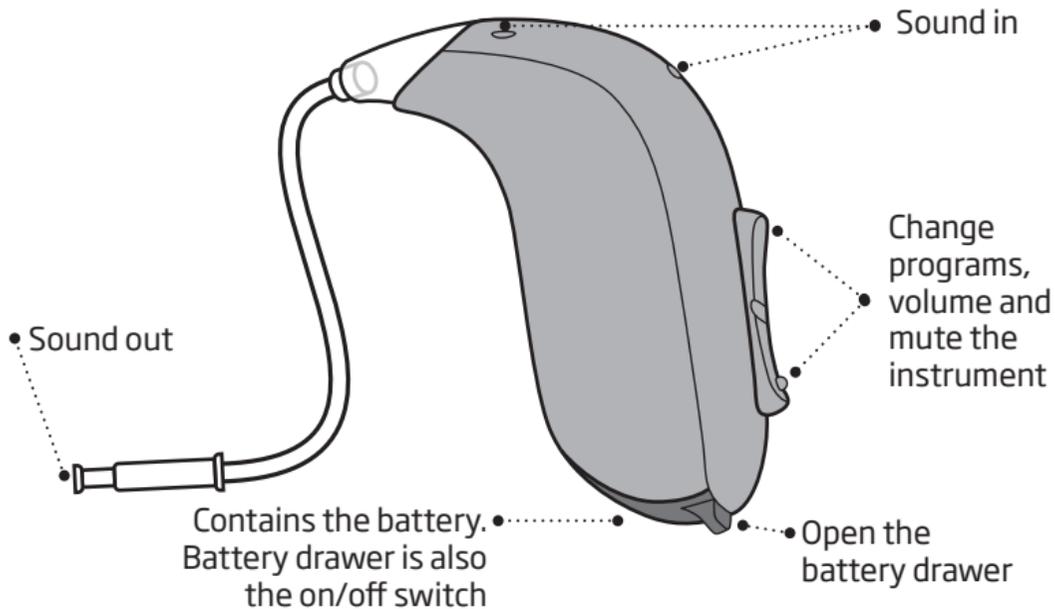

Identify left and right hearing instrument

It is important to distinguish between the left and the right instrument as they might be programmed differently.

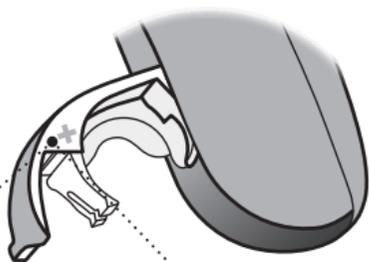


- A **BLUE** indicator inside the battery drawer marks the **LEFT** instrument.

- A **RED** indicator inside the battery drawer marks the **RIGHT** instrument

Battery (size 13)

Your hearing instrument is a miniature electronic device that runs on special batteries. To activate the hearing instrument, you must insert a new battery in the battery drawer. See how in the “Replace the battery” section.

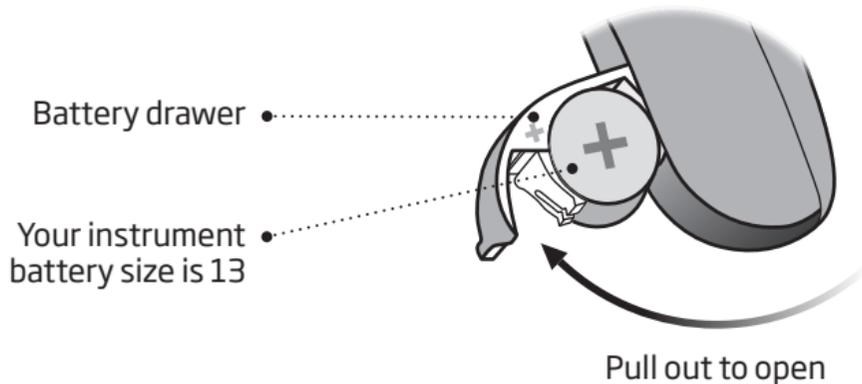


Turn the hearing instrument on and off

The battery drawer is also used to switch the hearing instrument on and off. To preserve the battery, make sure your instrument is switched off when you are not wearing it.


Turn ON

Close the battery drawer with the battery in place.

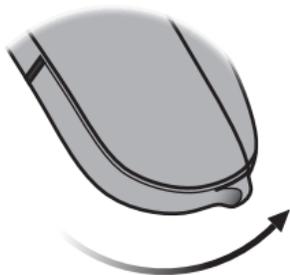

Turn OFF

Open the battery drawer

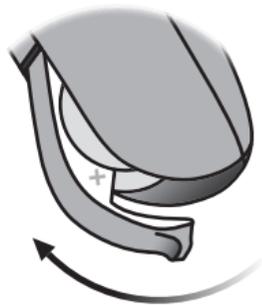

When to replace the battery

When it is time to replace the battery you will hear two beeps repeated in moderate intervals until the battery runs out.



Two beeps

= The battery is running low



Four beeps


= The battery has run out

Battery maintenance tip

To make sure the hearing instrument is always working bring spare batteries with you, or replace the battery before you leave home.


How to replace the battery

1. Remove

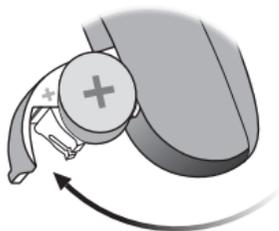


Fully open the battery drawer. Remove the battery.

2. Uncover

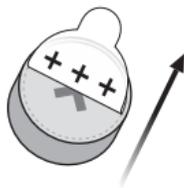


Remove the sticky label from the + side of the new battery.

3. Insert

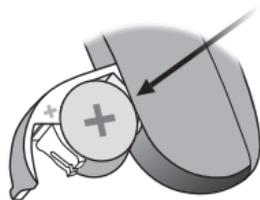

Insert the new battery into the battery drawer. Insert it from above, NOT from the side. Make sure the + side faces up.


4. Close

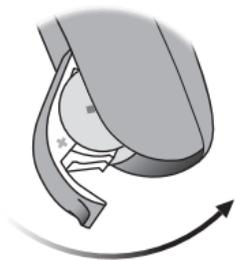

Close the battery drawer.
The instrument will play a
jingle through the ear piece.

Hold the ear piece close to
your ear to hear the jingle.

Tip

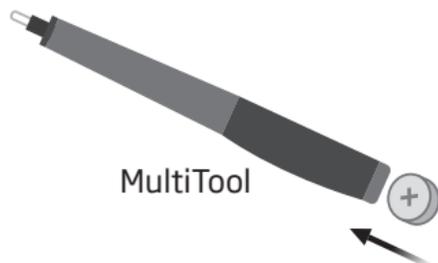

The MultiTool can be used for
battery change. Use the magnetic
end to remove and insert batteries.


The MultiTool is provided by
your Hearing Care Professional.

Caring for your hearing instrument

When handling your hearing instrument, hold it over a soft surface to avoid damage in case you drop it.

Clean the microphone openings

Carefully brush away debris from the openings. Gently brush the surface. Make sure the brush is not pressed in to the openings.

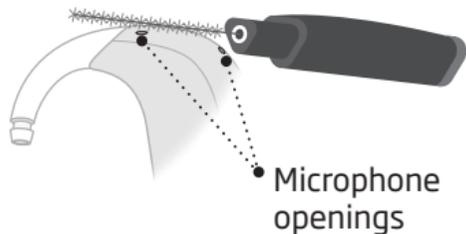


IMPORTANT NOTICE

Use a soft, dry cloth to clean the hearing instrument. It must never be washed or immersed in water or other liquids.

The MultiTool

The MultiTool contains a brush and a wire loop for cleaning ear wax from the mold. The brush can be replaced and purchased from your Hearing Care Professional.

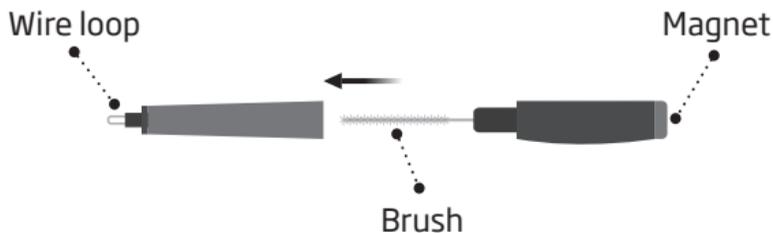

IMPORTANT NOTICE

Your MultiTool has a built-in magnet. Keep the MultiTool 30 cm away from credit cards and other magnetically sensitive devices.

Sound hook with mold

Read the following pages if you have a hearing instrument with a sound hook and a mold.

The mold is customized for you and fits your ear.



Put on the hearing instrument with mold

Step 1



Gently pull your ear outwards and press the mold in the direction of the ear canal, twisting slightly.


Step 2



Place the hearing instrument behind your ear.

Clean the mold

The mold should be cleaned regularly. Use a soft cloth to clean the surface of the mold. Use the MultiTool wire loop to clean the opening.

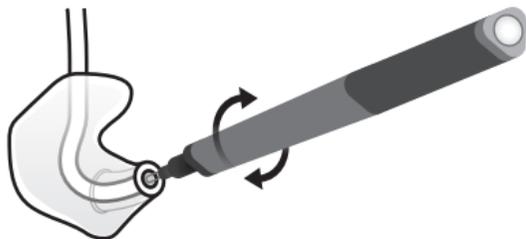

Replace the tube

The tubing between the mold and the instrument should be replaced when it becomes yellow or stiff. Consult your Hearing Care Professional in relation to this.

Wash the mold

Step 1




Make sure you are able to identify which mold is for which instrument before disconnecting them from the instruments.

Step 2



Disconnect the tube and mold from the hook. Hold the hook firmly, when pulling off the tube. Otherwise the instrument may be damaged.

Step 3

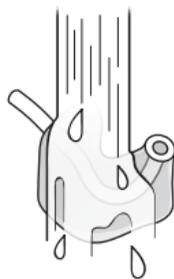


Wash the mold and tube using mild soap. Rinse and dry completely before reconnecting to the hearing instrument.

Thin tube (Corda miniFit)

Read the following pages if your instrument has a thin tube (Corda miniFit) with either a dome or Corda mold as ear piece.

Corda miniFit is a thin tube that brings the sound into your ear. The thin tube should always be used with an ear piece attached. Use only parts designed for your hearing instrument.

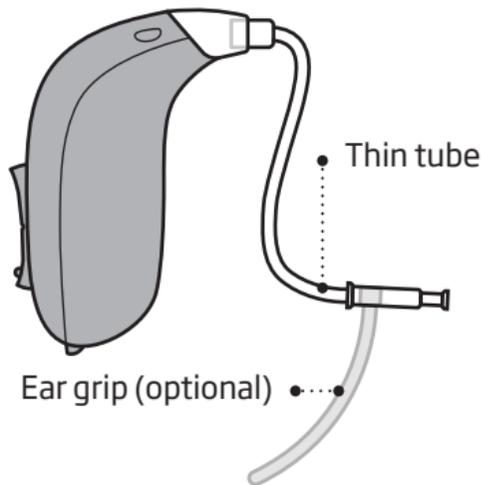


Put on the hearing instrument with thin tube

Step 1



Place the hearing instrument behind your ear.

Step 2

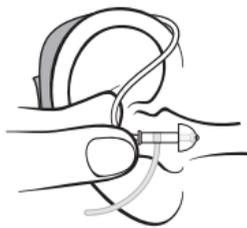

Hold the bend of the tube between your thumb and index finger. The ear piece should point towards the ear canal opening.

Step 3




Gently push the ear piece into your ear canal until the thin tube sits close against the side of your head.

Clean the thin tube


The thin tube should be cleaned to keep it clear from moisture and wax. If not cleaned it could become blocked with wax which means no sound will be able to come out.

Step 1

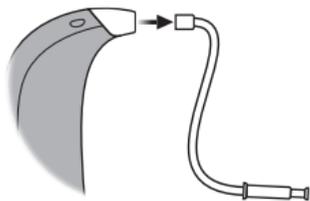


Pull the thin tube out of the hearing instrument

Step 2

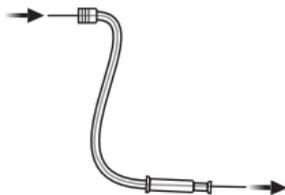

Push the cleaning tool all the way through the thin tube.

Step 3

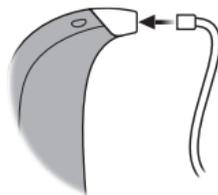

Reconnect the thin tube to the hearing instrument.

Thin tube with dome

There are 3 different types of domes. The domes are made from medically approved rubber material.

If you have a dome, check the type and size below.




Bass dome,
single vent



Bass dome,
double vent



Power dome

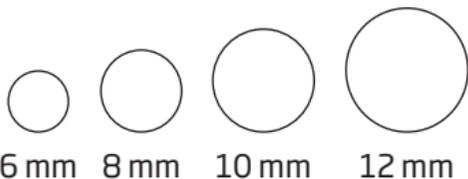


IMPORTANT NOTICE

If the dome is not on the tube when removed from the ear, it might still be in the ear canal. Consult your Hearing Care Professional for further instructions.

Replace the dome


The dome should not be cleaned. When the dome is more or less filled with wax, replace it with a new one.

Step 1

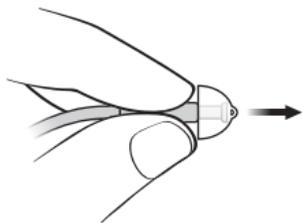


Hold on to the end of the thin tube and then pull off the dome.

Step 2

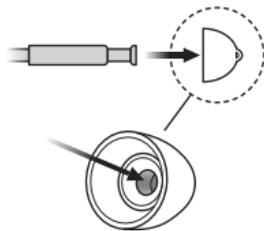

Place the thin tube exactly in the middle of the dome.

Step 3

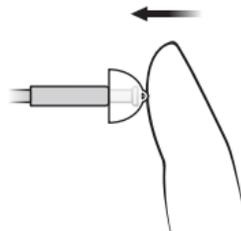

Push firmly to ensure that it is fastened securely.

Thin tube with Corda mold

Read the following pages if you have a Corda mold ear piece.
The Corda mold is customized for your ear shape.




Clean the Corda mold

The ear piece should be cleaned regularly.

The wax filter in the ear piece should be replaced when clogged or when the hearing instrument does not sound normal.

Always use the wax filter provided by your Hearing Care Professional.

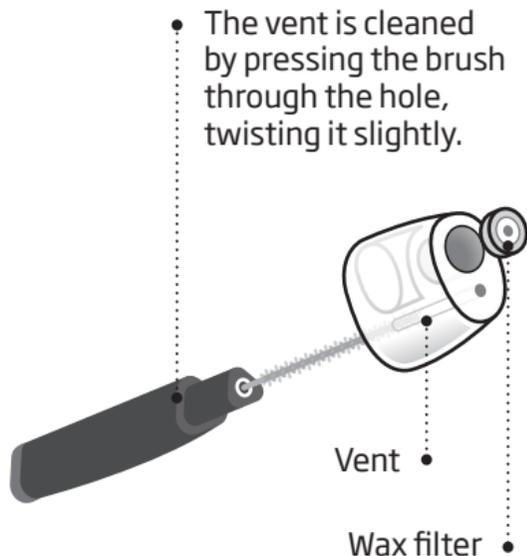

Optional features and accessories

The features and accessories described on the following pages are optional. Please contact your Hearing Care Professional to find out how your hearing instrument is programmed.

If you experience difficult listening situations, a special program may be helpful. These are programmed by your Hearing Care Professional.

Write down hearing situations where you may need help.

Mute the hearing instrument (optional)

Use the mute function if you need to silence the instrument while wearing it.




Apply a very long press to either end of the button to mute the instrument.
To reactivate the instrument, push the button briefly.

IMPORTANT NOTICE

Do not use the mute function as an off switch, as the hearing instrument still draws current from the battery in this mode.

Change volume (optional)

The push button allows you to adjust the volume.
You will hear a click when you turn the volume up or down.

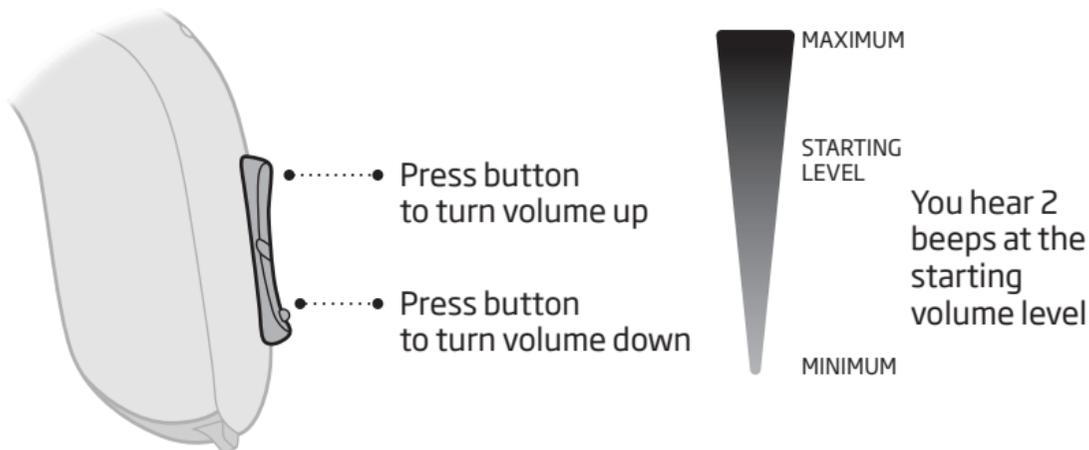


To be filled out by the Hearing Care Professional

Volume change	<input type="checkbox"/> LEFT	<input type="checkbox"/> RIGHT	<input type="checkbox"/> Short press
---------------	-------------------------------	--------------------------------	--------------------------------------


Quick reset

If you wish to return to the standard settings of the instrument programmed by your Hearing Care Professional, simply open and then close the battery drawer.

Open

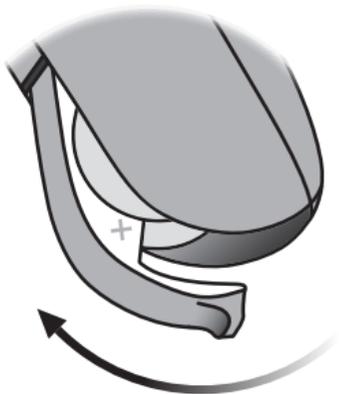


Close

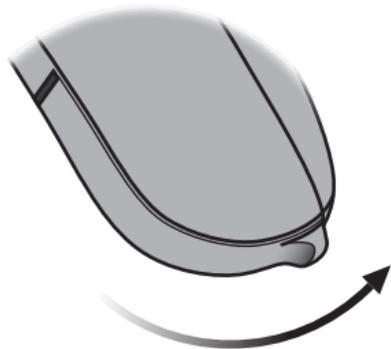

□ **Change programs** (optional)

Your hearing instrument can have up to 4 different programs. These are programmed by your Hearing Care Professional.

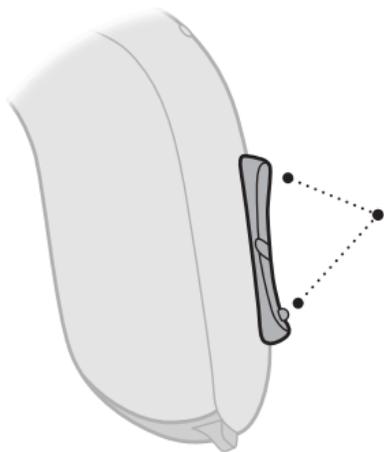

Press up or down
between programs

Note that you can change continuously between programs - both up and down in the program order. If for example you want to go from program 1 to 4, you can press the down button once instead of pressing the up button 3 times.

To be filled out by the Hearing Care Professional

Program	Sound you will hear when activated	When to use
1	 "1 beep"	
2	 "2 beeps"	
3	 "3 beeps"	
4	 "4 beeps"	

Program change:	<input type="checkbox"/> LEFT	<input type="checkbox"/> RIGHT	<input type="checkbox"/> Short press	<input type="checkbox"/> Long press
-----------------	-------------------------------	--------------------------------	--------------------------------------	-------------------------------------

Other options (optional)



Telecoil

Telecoil helps you hear better when using a telephone with a built-in loop or when you are in buildings with teleloop systems such as theaters, churches or lecture rooms. This symbol or a similar sign is shown wherever a teleloop has been installed.



Autophone

Autophone can automatically activate a phone program in the hearing instrument, if your telephone has a dedicated BTE13 PP magnet. Do not use any other magnet. The magnet needs to be placed on your telephone.

Direct Audio Input (DAI)

DAI allows your hearing instrument to receive signals directly from external sources such as TV, radio, music players, etc. The DAI adaptor is mounted to your instrument and via a cable connected to an external sound source.

For detailed information on these options contact your Hearing Care Professional.

IMPORTANT NOTICE

Only connect hearing aids with DAI to electrically powered devices that comply with EN 60950-1, EN 60065 or equivalent safety standards. The external signal source determines the safety of hearing aids with DAI.

Wireless accessories (optional)

As an enhancement to your wireless hearing instrument a broad range of wireless accessories are available.

ConnectLine

ConnectLine is a family of products that allows you to receive audio signals from TVs, phones, music players, PCs or an external microphone wirelessly through your hearing instrument.

Remote Control

The Remote Control offers an opportunity to change program, adjust the volume or mute your hearing instrument.

FM

Your hearing instrument supports the option of attaching an FM system. The FM system allows you to receive speech or audio signals directly to your hearing instrument from a dedicated transmitter.

For more detailed information on wireless accessories contact your Hearing Care Professional or visit **www.oticonusa.com**.

Tamper-resistant battery drawer (optional)


To keep the battery out of reach of infants, small children and people with learning difficulties, a tamper-resistant battery drawer should be used.

IMPORTANT NOTICE

Avoid excessive force to open the battery drawer when in the locked position. Do not force the battery drawer beyond its fully opened position. Make sure to insert the battery correctly.


Please contact your Hearing Care Professional if you suspect the tamper-resistant effectiveness and locking ability are damaged.

To lock the battery drawer

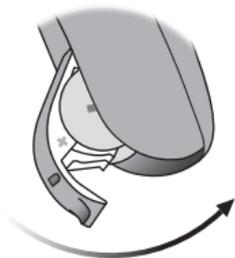


Close the battery door completely and make sure it is locked.

Tool to unlock the battery drawer

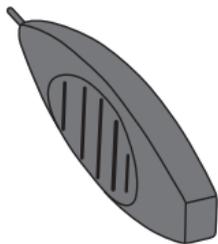

Use this small tool provided by your Hearing Care Professional.

Unlock the battery drawer

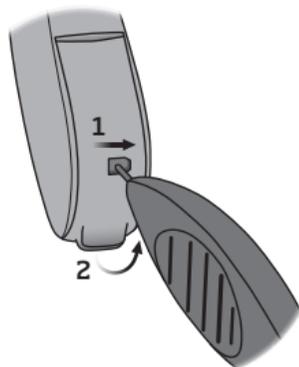

Insert the tip of the tool into the small hole at the back of the battery drawer.

1. Move it to the right and hold it
2. Open the battery drawer

Warnings

You should familiarize yourself fully with the following general warnings before using your hearing instrument for personal safety and to secure correct use.

Please note that a hearing aid will not restore normal hearing and will not prevent or improve a hearing impairment resulting from organic conditions. Furthermore, note that in most cases, infrequent use of a hearing instrument does not permit a user to attain full benefit from it.

Consult your Hearing Care Professional if you experience unexpected operations or events with your hearing instrument.

Usage of hearing instruments

- Hearing instruments should be used only as directed and adjusted by your Hearing Care Professional. Misuse can result in sudden and permanent hearing loss.
- Never allow others to wear your hearing instrument as incorrect use could cause permanent damage to their hearing.

Choking hazards & risk of swallowing batteries

- Hearing instruments, their parts, and batteries should be kept out of reach of children and anyone who might swallow these items, or otherwise cause injury to themselves.

- Batteries have occasionally been mistaken for pills. Therefore, check your medicine carefully before swallowing any pills.
- Most hearing instruments can be supplied with a tamper-resistant battery drawer upon request. This is strongly recommended for infants, small children and people with learning difficulties.
- Children below 36 months must always use a tamper-resistant battery drawer. This option is available in BTE, miniBTE, RITE and miniRITE instruments.

If a battery or hearing instrument is swallowed, see a doctor immediately and contact the national Poison Center at 1-800-222-1222 or the national Battery ingestion Hotline at 1-202-625-3333.

Battery use

- Always use batteries recommended by your Hearing Care Professional. Batteries of low quality may leak and cause bodily harm.
- Never attempt to recharge your batteries and never dispose of batteries by burning them. There is a risk that the batteries will explode.

Warnings

Dysfunction

- Be aware of the possibility that your hearing instrument may stop working without notice. Keep this in mind when you depend on warning sounds (e.g. when you are in traffic). The hearing instruments may stop functioning, for instance if the batteries have expired or if the tubing is blocked by moisture or ear wax.

Active implants

- Caution must be taken with active implants. In general, follow the guidelines recommended by manufacturers of implantable defibrillators and pacemakers regarding use with mobile phones and magnets.
- Your hearing instrument has a wireless transmission. Please keep the hearing instrument at least 15 cm away from the implant, e.g. do not carry it in a breast pocket.
- When the hearing instruments are mounted on or in the ears, the distance between the hearing instruments and the implant is larger than 15 cm.
- Your Autophone magnet and Multi-Tool (which has a built-in magnet) should be kept more than 30 cm away from the implant e.g. do not carry it in a breast pocket.

- If you have an active brain implant, please contact the manufacturer of your implantable device for information about the risk of disturbance.

Explosives

- The power source in your hearing instrument has insufficient energy to cause fire in normal usage conditions. The hearing instrument has not been tested for compliance with international standards concerning explosive environments. It is recommended not to use your hearing instrument in areas where there is a danger of explosions.

X-ray, CT, MR, PET scanning and electrotherapy

- Remove your hearing instrument for example during X-ray, CT/MR/PET scanning, electrotherapy or surgery as your hearing instrument may be damaged when exposed to strong fields.

Warnings

Avoiding heat and chemicals

- Your hearing instrument must never be exposed to extreme heat e.g. left inside a parked car in the sun.
- Your hearing instrument must not be dried in microwave ovens or other ovens.
- The chemicals in cosmetics, hairspray, perfume, aftershave lotion, suntan lotion and insect repellent can damage your hearing instrument. Always remove your hearing instrument before applying such products and allow time to dry before putting it on.

Power instrument

- This is a power instrument and special care should be exercised in selecting, fitting and using a hearing instrument where maximum sound pressure capability exceeds 132 dB SPL (IEC 711), as there may be risk of impairing the remaining hearing of the hearing instrument user.

Possible side effects

- Hearing instruments, molds or domes may cause an accelerated accumulation of ear wax.
- The otherwise non-allergenic materials used in hearing instruments may in rare cases cause a skin irritation or any other unusual condition.

Please seek consultation with a physician if these conditions occur.

Interference

- Your hearing instrument has been thoroughly tested for interference, according to the most stringent international standards. However, interference with your hearing instrument and other devices may occur, (e.g. some mobile telephones, citizens band systems and shop alarm systems). If this occurs, increase the distance between the hearing instrument and the device.

Warnings

Warning to hearing instrument dispensers

A hearing instrument dispenser should advise a prospective hearing instrument user to consult immediately with a licensed physician (preferably an ear specialist) before dispensing a hearing instrument if the hearing instrument dispenser determines through inquiry, actual observation, or review of any other available information concerning the prospective user, that the prospective user has any of the following conditions:

- (i) Visible congenital or traumatic deformity of the ear.
- (ii) History of active drainage from the ear within the previous 90 days.
- (iii) History of sudden or rapidly progressive hearing loss within the previous 90 days.
- (iv) Acute or chronic dizziness.
- (v) Unilateral hearing loss of sudden or recent onset within the previous 90 days.
- (vi) Audiometric air-bone gap equal to or greater than 15 decibels at 500 Hertz (Hz), 1,000 Hz, and 2,000 Hz.
- (vii) Visible evidence of significant cerumen accumulation or a foreign body in the ear canal.
- (viii) Pain or discomfort in the ear.

Special care should be exercised in selecting and fitting a hearing instrument whose maximum sound pressure capability exceeds 132 dB SPL as there may be risk of impairing the remaining hearing of the hearing instrument user.

Important notice for prospective hearing instrument users

- Good health practice requires that a person with a hearing loss have a medical evaluation by a licensed physician (preferably a physician who specializes in diseases of the ear) before purchasing a hearing instrument. Licensed physicians who specialize in diseases of the ear are often referred to as Otolaryngologists, Otologists or Otorhinolaryn-

gologists. The purpose of medical evaluation is to ensure that all medically treatable conditions that may affect hearing are identified and treated before the hearing instrument is purchased. Following the medical evaluation, the physician will give you a written statement that states that your hearing loss has been medically evaluated and that you may be considered a candidate for a hearing instrument. The physician will refer you to an audiologist or a hearing instrument dispenser, as appropriate, for a hearing instrument evaluation.

Warnings

- The audiologist or hearing instrument dispenser will conduct a hearing instrument evaluation to assess your ability to hear with and without a hearing instrument. The hearing instrument evaluation will enable the audiologist or dispenser to select and fit a hearing instrument to your individual needs. If you have reservations about your ability to adapt to amplification, you should inquire about the availability of a trial, rental or purchase-option program. Many hearing instrument dispensers now offer programs that permit you to wear a hearing instrument for a period of time for a nominal fee, after which you may decide if you want to purchase the hearing instrument. Federal law limits the sale of hearing instruments to those individuals who have obtained a medical evaluation from a licensed physician.
- Federal law permits a fully informed adult to sign a waiver statement declining the medical evaluation for religious or personal beliefs that preclude consultation with a physician. The exercise of such a waiver is not in your best health interest and its use is strongly discouraged. A hearing instrument will not restore normal hearing and will not prevent or improve a hearing impairment resulting from organic

conditions. A hearing instrument is only part of hearing rehabilitation and may need to be supplemented by auditory training and lip reading.

Children with hearing loss

In addition to seeing a physician for medical evaluation, a child with a hearing loss should be directed to an audiologist for evaluation and rehabilitation, since hearing loss may cause problems in language development and educational and social growth of a child. An audiologist is qualified by training and experience to assist in the evaluation and rehabilitation of a child with a hearing loss. If the user is an infant, small child, or person of mental incapacity, it is recommended that the hearing instrument be modified with a tamper-resistant battery compartment.

Troubleshooting guide

Symptom	Possible causes
No sound	Worn-out battery
	Clogged tube or earpieces (mold, Corda mold or dome)
Intermittent or reduced sound	Clogged sound outlet
	Moisture
	Worn-out battery
	Clogged damping filter in sound hook
Squealing noise	Hearing instrument earpiece inserted incorrectly
	Ear wax accumulated in ear canal

If none of the above solutions works, consult your Hearing Care Professional for assistance.

Solutions

Replace the battery

Clean mold

Clean Corda miniFit (thin tube), replace dome or replace wax filter

Clean mold

Clean Corda miniFit (thin tube), replace dome or replace wax filter

Wipe battery with a dry cloth and/or dry tube

Replace the battery

Contact your Hearing Care Professional for assistance

Re-insert earpiece (mold, Corda mold or dome)

Have ear canal examined by your doctor

Water resistant (IP58)

Your hearing instrument is water resistant, which means it is designed to be worn in all daily life situations. Therefore, you do not have to worry about sweat or getting wet in the rain. Should your hearing instrument come in contact with water and stop working, please follow these guidelines:

1. Gently wipe off any water
2. Open the battery drawer and remove the battery and gently wipe off any water in the battery drawer
3. Let the hearing instrument dry with the battery drawer left open for approximately 30 minutes
4. Insert a new battery

IMPORTANT NOTICE

Do not wear your hearing instrument while showering or participating in water activities. Do not immerse your hearing instrument in water or other liquids.

Conditions

Operating conditions	Temperature: +1°C to +40°C Relative humidity: 5% to 93%, non-condensing
Storage and transportation conditions	Temperature and humidity shall not exceed the limits below for extended periods during transportation and storage: Temperature: -25°C to +60°C Relative humidity: 5% to 93%, non-condensing

Warranty certificate

Name of owner: _____

Dispenser: _____

Dispenser address: _____

Dispenser phone: _____

Purchase date: _____

Warranty period: _____ Month: _____

Model left: _____ Serial no.: _____

Model right: _____ Serial no.: _____

Battery size: _____

International warranty

Your hearing instrument is covered by an international limited warranty issued by the manufacturer for a period of 12 months from the date of delivery. This limited warranty covers manufacturing and material defects in the hearing instrument itself, but not accessories such as batteries, tubing, ear wax filters, etc. Problems arising from improper handling or care, excessive use, accidents, repairs made by an unauthorized party, exposure to corrosive conditions, physical changes in your ear, damage due to foreign objects entering the device, or incorrect adjustments are NOT covered by the limited warranty and may void it. The above warranty does not affect any legal rights that you might have under

applicable national legislation governing the sale of consumer goods. Your Hearing Care Professional may have issued a warranty that goes beyond the clauses of this limited warranty. Please consult him/her for further information.

If you need service

Take your hearing instrument to your Hearing Care Professional, who may be able to sort out minor problems and adjustments immediately.

Mobile phone

Some hearing instrument users have reported a buzzing sound in their hearing instrument when they are using mobile phones, indicating that the mobile phone and hearing instrument may not be compatible.

The ANSI C63.19 standard determines the prediction of compatibility between a specific hearing instrument and a mobile phone by: adding the numerical value of the rating for the hearing instrument immunity to the numerical value of the rating for the mobile phone emissions. A sum of 4 would indicate that the combination of wireless device and hearing instrument is usable; a combined rating that equals at least 5 would provide normal use;

a combined rating of 6 or greater would indicate excellent performance.

Whereas all hearing instruments have acoustic coupling, only the larger instruments have the physical space for telecoil (inductive) coupling. These two types of coupling have different rating scales (M1-M4 for acoustic coupling and T1-T4 for telecoil coupling, respectively) and both ratings are therefore relevant when predicting the compatibility of a particular hearing instrument.

For a hearing instrument with both acoustic coupling and telecoil coupling with a rating of M4/T2 and with a telephone rating of M3/T3), the combined

rating is 7 (M4 + M3) for the acoustic coupling and 5 (T2 + T3) for the telecoil coupling. According to the guidelines given above, both types of coupling will thereby be acceptable, with the acoustic coupling indicating excellent performance and the telecoil coupling indicating normal use.

The above equipment performance measurements, categories and system classifications are based upon the best information available, but it cannot be guaranteed that all users will be satisfied.

The immunity of this hearing instrument is M4/T4.

IMPORTANT NOTICE

The performance of individual hearing instruments may vary with individual mobile phones. Therefore, please try this hearing instrument with your mobile phone or, if you are purchasing a new phone, be sure to try it with your hearing instrument prior to purchase.

For additional guidance, please ask your mobile phone provider for the booklet entitled "Hearing Aid Compatibility with Digital Wireless Cell Phones."

Technical information

The hearing instrument contains a radio transmitter using short range magnetic induction technology working at 3.84 MHz. The magnetic field strength of the transmitter is $< -42 \text{ dB}\mu\text{A/m @ 10m}$.

The emission power from the radio system is well below international emission limits for human exposure. For comparison, the radiation of the hearing instrument is lower than unintended electromagnetic radiation from, for example, halogen lamps, computer monitors, dishwashers, etc. The hearing instrument complies with international standards concerning Electromagnetic Compatibility.

Due to the limited space available on the instruments, all relevant approval markings are found in this document.

Additional information can be found in the Product Information sheet on www.oticonusa.com.

The hearing instrument contains a module with:

FCC ID: U28FU2BTEPP
IC: 1350B-FU2BTEPP

The device complies with Part 15 of the FCC rules and RSS-210 of Industry Canada.

Operation is subject to the following two conditions:

1. This device may not cause harmful interference.
2. This device must accept any interference received, including interference that may cause undesired operation.

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Oticon declares that this hearing instrument is in compliance with the essential requirements and other relevant provisions of Directive 1999/5/EC.

Declaration of Conformity is available from the manufacturer.



Manufactured by:
Oticon A/S
Kongebakken 9
DK-2765 Smørum
Denmark
www.oticon.com

CE 0543 0682



Waste from electronic equipment must be handled according to local regulations.



E2831

IP58

Settings overview for your hearing instrument

Left			Right	
<input type="checkbox"/> Yes	<input type="checkbox"/> No	Volume control	<input type="checkbox"/> Yes	<input type="checkbox"/> No
<input type="checkbox"/> Yes	<input type="checkbox"/> No	Program shift	<input type="checkbox"/> Yes	<input type="checkbox"/> No
<input type="checkbox"/> Yes	<input type="checkbox"/> No	Mute	<input type="checkbox"/> Yes	<input type="checkbox"/> No
Volume control indicators				
<input type="checkbox"/> On	<input type="checkbox"/> Off	Beeps at min /max volume	<input type="checkbox"/> On	<input type="checkbox"/> Off
<input type="checkbox"/> On	<input type="checkbox"/> Off	Clicks when changing volume	<input type="checkbox"/> On	<input type="checkbox"/> Off
<input type="checkbox"/> On	<input type="checkbox"/> Off	Beeps at preferred volume	<input type="checkbox"/> On	<input type="checkbox"/> Off
Battery indicators				
<input type="checkbox"/> On	<input type="checkbox"/> Off	Low battery warning	<input type="checkbox"/> On	<input type="checkbox"/> Off

TECHNICAL DATA

BTE13 PP

Alta2 Pro, Alta2

oticon
PEOPLE FIRST

BTE13 PP**105****0 dB SPL ref. 20 mPa**

ZCC Coupler
measured
according to
American National
Standard
ANSI S 3.22 (2003)
and S3.7 (1995).

Supply voltage:
Battery Zinc
Air 1.4 Volt

Peak OSPL90		133 (131*) dB SPL
HF Average OSPL90		128 (120*) dB SPL
Peak Full-on Gain		67 (67*) dB
HF Average Full-on Gain		63 (55*) dB
Reference Test Gain		48 dB
Frequency Range		100-5700 Hz
Total Harmonic Distortion	500 Hz	2 %
Total Harmonic Distortion	800 Hz	<2%
Total Harmonic Distortion	1600 Hz	<2%
Battery Consumption		1.3 mA
Equivalent Input Noise Level (omni/dir)		14/28 dB SPL
HF Average SPLITS (left/right ear)		105/105 dB SPL
Attack Time		1.6 ms
Release Time		118 ms

* For instruments fitted with thin tube


BTE13 PP

105

OSPL90 - Output Sound Pressure Level

Input: 90 dB SPL.

Technical setting: A0

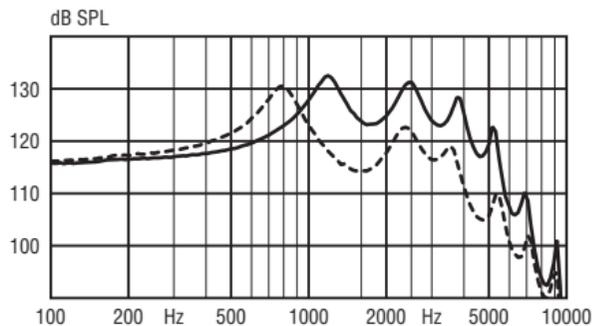


Full-on Gain

Input: 50 dB SPL.

Technical setting: A0

—— Sound Hook

----- Thin tube
(size 1/1.3)

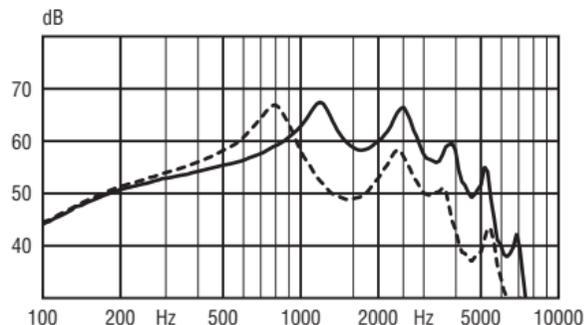

TECHNICAL DATA

BTE13 PP

Nera2 Pro, Nera2, Ria2 Pro, Ria2

oticon
PEOPLE FIRST

BTE13 PP**105****0 dB SPL ref. 20 mPa**

ZCC Coupler
 measured
 according to
 American National
 Standard
 ANSI S 3.22 (2003)
 and S3.7 (1995).

Supply voltage:
 Battery Zinc
 Air 1.4 Volt

Peak OSPL90	133 (131*) dB SPL
HF Average OSPL90	128 (120*) dB SPL
Peak Full-on Gain	67 (67*) dB
HF Average Full-on Gain	63 (55*) dB
Reference Test Gain	48 dB
Frequency Range	100-5700 Hz
Total Harmonic Distortion 500 Hz	2 %
Total Harmonic Distortion 800 Hz	<2%
Total Harmonic Distortion 1600 Hz	<2%
Battery Consumption	1.3 mA
Equivalent Input Noise Level (omni/dir)	14/28 dB SPL
HF Average SPLITS (left/right ear)	105/105 dB SPL
Attack Time	1.6 ms
Release Time	118 ms

* For instruments fitted with thin tube


BTE13 PP

105

OSPL90 - Output Sound Pressure Level

Input: 90 dB SPL.

Technical setting: A0

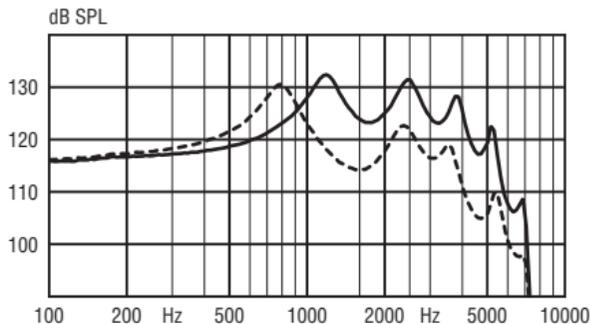


Full-on Gain

Input: 50 dB SPL.

Technical setting: A0

—— Sound Hook

----- Thin tube
(size 1/1.3)

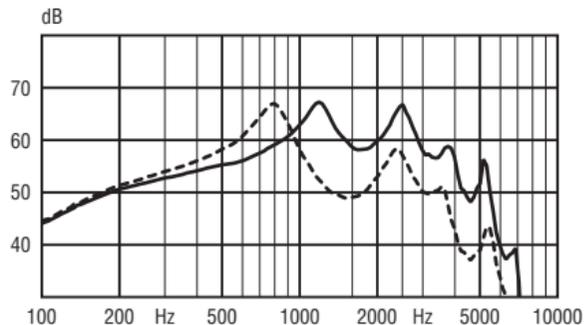

People First

People First is our promise
to empower people
to communicate freely,
interact naturally and
participate actively



0000161785000001

oticon
PEOPLE FIRST