
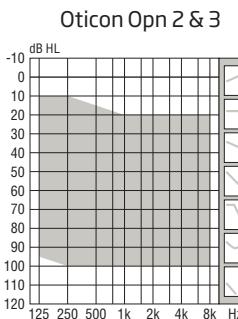


Technical data sheet

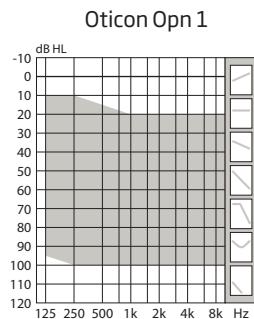

100

General features:

- Digital Programmable
- Automatic or Manual Volume Control
- Maximum Output Control System
- MPO-Maximum Power Output
- GC-Gain Control
- AGC-Automatic Gain Control
- Noise Reduction
- Feedback Management
- Dual Microphone
- 4 Programs (when push button is selected)

	Oticon Opn 1	Oticon Opn 2	Oticon Opn 3
Speech Understanding			
OpenSound Navigator™	Level 1	Level 2	Level 3
- Balancing power effect	100%	50%	50%
- Max. noise removal	9 dB	5 dB	3 dB
Speech Guard™ LX	Level 1	Level 2	Level 3
Spatial Sound™ LX	4 estimators	2 estimators	2 estimators
Soft Speech Booster LX	•	•	•
Speech Rescue™ LX	•	•	•
Sound Quality			
Clear Dynamics	•	•	-
Spatial Noise Management	•	•	-
Fitting Bandwidth*	10 KHz	8 KHz	8 KHz
Processing Channels	64	48	48
Bass Boost (streaming)	•	•	•
Listening Comfort			
Transient Noise Management	4 configurations	On/Off	On/Off
Feedback shield LX	•	•	•
Wind Noise Management	•	•	•
Binaural Coordination***	•	•	•
Personalisation & Optimising Fitting			
YouMatic™ LX	3 configurations	2 configurations	1 configuration
Fitting Bands	16	14	12
Multiple Directionality Options	•	•	•
Adaptation Management	•	•	•
Oticon Firmware Updater	•	•	•
Fitting Formulas	VAC+, NAL-NL1+2, DSL v5.0	VAC+, NAL-NL1+2, DSL v5.0	VAC+, NAL-NL1+2, DSL v5.0
Acoustic Notifications	•	•	•
Connecting to the World			
Stereo streaming (2.4 GHz)	○	○	○
Oticon ON App	○	○	○
ConnectClip	○	○	○
Remote Control 3.0	○	○	○
TV Adapter 3.0	○	○	○
Autophone	○	○	○
Tinnitus SoundSupport™***	•	•	•
Battery life, hours**	50-60 / 90-115	50-60 / 90-115	50-60 / 90-115

* Bandwidth accessible for gain adjustments during fitting


** Battery size 312 - IEC PR41 / Battery size 13 - IEC PR48.

Real usage battery life is shown as an estimated interval based on mixed use cases with variable amplification settings and variable input levels, incl. direct stereo streaming from a TV (25% of the time) and streaming from a mobile phone (6% of the time).

*** If push button is chosen

• Default ○ Optional - Not included

OTICON | Opn
ITC, ITE HS & FS 100

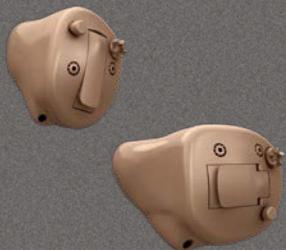

Oticon Opn™ ITC,ITE HS & FS introduce an updated faceplate design.

OpenSound Navigator™ provides better speech understanding by continuously analysing the environment, balancing all sound sources and attenuating the dominating noise.

TwinLink™ wireless technology combines binaural communication and 2.4 GHz connectivity in stereo directly to external digital devices with very low power consumption. 2.4 GHz is an optional.

Oticon Opn is a Made for iPhone® hearing aid.

Oticon Opn is built on the Velox™ platform, providing frequency resolution in 64 channels (Opn 1).

Fully programmable with updatable firmware, the Velox platform is ready for the future.



Made for
iPhone | iPad | iPod

IP68

For information on compatibility, please visit www.oticon.global/connectivity.


oticon
PEOPLE FIRST

Technical data			Ear Simulator			2CC Coupler		
Measured according to			IEC 60118-0:1983/AMD1:1994, IEC 60118-0:2015, IEC 60118-1:1995+AMD1:1998 CSV and IEC 60318-4:2010			ANSI S3.22-2014, IEC 60118-0:2015 and IEC 60318-5:2006		
Oticon Opn ITC ITE HS & FS 100			Opn 1	Opn 2	Opn 3	Opn 1	Opn 2	Opn 3
Frequency range Hz			100-7500	100-7500	100-7500	100-7100	100-7100	100-7100
MPO-OSPL90	Peak			134 dB SPL			125 dB SPL	
	1600 Hz			133 dB SPL			125 dB SPL	
	HFA-OSPL90			130 dB SPL			122 dB SPL	
Full-on gain*	Peak			72 dB			63 dB	
	1600 Hz			68 dB			60 dB	
	HFA-FOG			67 dB			58 dB	
Reference test gain				58 dB			45 dB	
Telecoil output (1600 Hz)	1 mA/m field			98 dB SPL			-	
	10 mA/m field			118 dB SPL			-	
	SPLITS L/R			-			103/103 dB SPL	
Total harmonic distortion (Input 70 dB SPL)	500 Hz			2 %			<2 %	
	800 Hz			2 %			<2 %	
	1600 Hz			3 %			<2 %	
Equivalent input noise level	Omni			14 dB SPL			15 dB SPL	
	Dir			26 dB SPL			28 dB SPL	
Battery consumption**	Typical			1.8 mA			1.8 mA	
	Quiescent			1.7 mA			1.7 mA	
Battery life, calculated, hours 312 and 13***				105 / 175			100 / 170	
IRIL (IEC 60118-13:2016)				700/1400/2000 MHz: 19/12/6 dB SPL				

* Measured with the gain control of the hearing aid set to its full-on position minus 20 dB and with an input SPL of 70 dB. This is to obtain a gain response equal to the full-on gain response from e.g. IEC 60118-0+A1:1994 but without influence of feedback.

** Battery current is measured according to IEC 60118-0:1983/AMD1:1994 §7.11, IEC 60118-0:2015 §7.7 and ANSI S3.22:2014 §6.13 after a settling time of minimum 3 minutes.

*** Based on the standardised battery consumption measurement (IEC 60118-0:1983/AMD1:1994). The actual battery life depends on battery quality, use pattern, active feature set, hearing loss and sound environment.

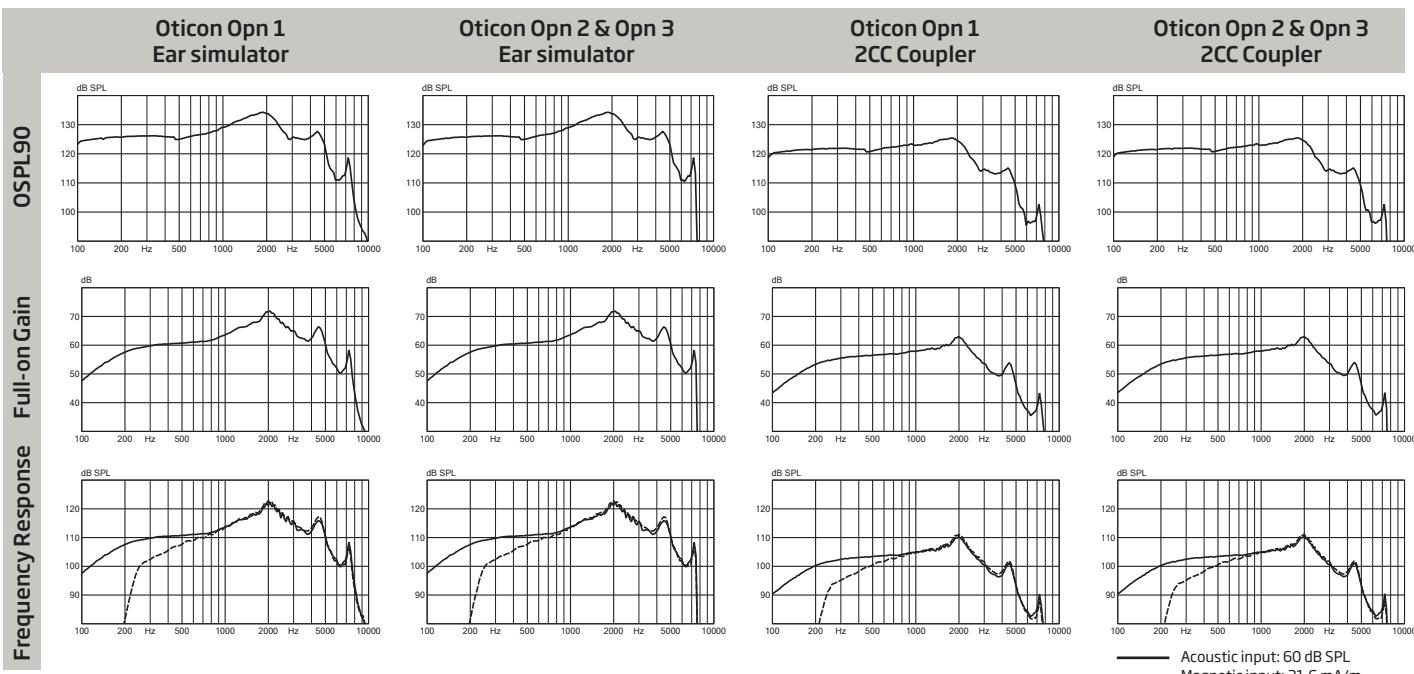

Technical information: Omnidirectional mode is used unless otherwise stated.

Operating conditions	Storage and transportation conditions	Instrument warning
Temperature: +1°C to +40°C	Temperature and humidity should not exceed the following limits for extended periods during transportation and storage.	The maximum output capability of the hearing instrument may exceed 132 dB SPL (IEC 711). Special care should be exercised in selecting and fitting the instrument as there may be risk of impairing the remaining hearing of the hearing aid user.
Relative humidity: 5% to 93%, non-condensing	Temperature: -25°C to +60°C Relative humidity: 5% to 93%, non-condensing	